

# Wound Care Formulary

This document is also available in other languages and formats upon request.

Su richiesta, questo documento è disponibile in altre lingue e in altri formati.

Sur demande, ce document peut être fourni en d'autres langues et formats.

Na życzenie, dokument ten można uzyskać w innych językach i formatach.

यह दस्तावेज़ अनुरोध किए जाने पर अन्य भाषाओं और प्रारूपों में उपलब्ध है।

ਇਹ ਦਸਤਾਵੇਜ਼ ਬੇਨਤੀ ਕੀਤੇ ਜਾਣ ਤੇ ਹੋਰ ਭਾਸ਼ਾਵਾਂ ਅਤੇ ਰੂਪਾਂ ਵਿੱਚ ਉਪਲਬਧ ਹੈ।

در صورت درخواست این سند به زبانها و شکلهای مختلف در اختیار شما قرار می گیرد.

یہ دستاویز دیگر زبانوں اور مطلوبہ شکلوں (فارمیٹ) میں بھی دستیاب ہے

هذه الوثيقة متاحة بلغات أخرى وباشكال غير الكتابة المقروءة وذلك عند الطلب

**The Tissue Viability Team**  
C/O Ashfield Health and Wellbeing Centre  
Portland Street  
Kirkby-in-Ashfield  
Nottinghamshire  
NG17 7AE

**01623 784759**

[www.nottinghamshirehealthcare.nhs.uk](http://www.nottinghamshirehealthcare.nhs.uk)



NH787

February 2019



Version 12  
02/19

**positive**  
about integrated healthcare

## Irrigation Solution

Proprietary Name	Sizes ml	NHS Logistics Code	Pack Size
Irripod	20ml pod	MRB742	25
Normasol (sachet)	100ml	MRB357	10
	25ml	MRB358	25

## Gauze Swabs, Basic Bandages, Tubular Bandages and Tapes

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Clinifast</b>			
<b>Small Limb (Red Line) 3.5cm</b>	1m	EGP061	Single
	10m	EGP018	Single
<b>Medium Limb (Green Line) 5.0cm</b>	1m	EGP058	Single
	5m	EGP054	Single
<b>Large Limb (Blue Line) 7.5cm</b>	1m	EGP167	Single
	5m	EGP054	Single
<b>Trunk (Child) (Yellow Line) 10.75cm</b>	3m	EGP162	Single
	5m	EGP056	Single
<b>Trunk (Adult) (Beige Line) 17.5cm</b>	10m	EGP022	Single

Ko-Flex	Pg 5
Ko-Flex Long	Pg 5
K-Plus (latex free)	Pg 5
K-Plus Long (latex free)	Pg 5
K-Three C (latex free)	Pg 5
K-Two	Pg 4
K-Two Reduced	Pg 4
Kerrapro	Pg 14
Knitband	Pg 3

Lohmann-rauscher (formally Activa) Pg 4

Medi	Pg 4
Medihoney	Pg 17
Medi-Derma S	Pg 9

N/A Ultra	Pg 6
Normasol	Pg 2
KCI Activac dressings	Pg 20

Oakmed Wound Pouch	Pg 15
Octenilin	Pg 18

Permafoam (trachi-dress)	Pg 12
Polymem	Pg 18

Silflex	Pg 16
Softdrape	Pg 3
Softpore	Pg 7
Sorbion Sachet Border	Pg 13
Sterile Gauze	Pg 3

Tegaderm Hydrocolloid	Pg 10
Tegaderm IV	Pg 8
Telfa	Pg 6

Urgotul Pg 16

Venturi Dressings	Pg 21
Viscopaste	Pg 4

Xupad Pg13

Zetuvit E Sterile Pg 12

## Index of Dressings

Actico	Pg 4
Allevyn Gentle Adh	Pg 16
Allevyn N/A	Pg 12
Allevyn Adhesive	Pg 12
Appeel	Pg 6
Aquacel	Pg 11
Aquacel Extra	Pg 11
Aquacel AG	Pg 18
Atrauman	Pg 6
Atruman AG	Pg 15
Bastios Viegas	Pg 13
Clearfilm	Pg 8
Clinipore	Pg 5
Clinisorb	Pg 14
Cutimed Protect	Pg 9
Cutimed Siltec	Pg 16
Cutimed Siltec L	Pg 7
Cutimed Sorbact	Pg 19
Clinifast all sizes	Pg 2
Debrisoft	Pg 19
Duoderm Extra Thin	Pg 10
Duoderm Signal	Pg 10
Eclipse	Pg 13
Flaminal	Pg 19
Hydrogel Aquaform	Pg 10
Hydrogel sheet	Pg 10
Hydrofilm Plus	Pg 8
Hypafix	Pg 5
Inadine	Pg 15
Iodosorb	Pg 17
Iodosorb Paste	Pg 17
Irripod	Pg 2
Juzo	Pg 4
K-Four	Pg 4
K-Lite	Pg 3
K-Soft	Pg 3

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Sterile Dressing Packs</b> <b>Softdrape Wound Care Packs</b>	Small Gloves	EJA045	20
	Medium Gloves	EJA046	20
	Large Gloves	EJA047	20
<b>Sterile Gauze Pack of 5</b>	7.5cm x 7.5cm	ENK140	25
	10cm x 10cm	ENK143	25
<b>Orthopaedic Wool</b> <b>Formflex Natural</b> <b>Lantor-non Sterile</b>	5cm x 2.7m	EPA029	6
	7.5cm x 2.7m	EPA030	6
	10cm x 2.7m	EPA031	6
	15cm x 2.7m	EPA032	6
	20cm x 2.7m	EPA033	6
<b>K-Soft</b> <b>K-Soft Long</b>	10cm x 3.5m	EPA028	24
	10cm x 4.5m	EPA174	24
<b>Retention Bandage</b> <b>Knitband</b>	5cm x 4m	EDB115	25
	7cm x 4m	EDB116	25
	10cm x 4m	EDB117	25
	15cm x 4m	EDC089	100
<b>K-Lite</b>	5cm x 4.5m	ECA084	16
	7cm x 4.5m	ECA086	16
	10cm x 4.5m	ECA100	16
	15cmx4.5m	ECA109	16
<b>K-Lite Long</b> * use 10cm for lower limb bandaging	10cm x 5.25m	ECA173	16

## Hosiery Formulary

The following hosiery items have been evaluated by the leg ulcer coordinators and should be prescribed where possible—for prescribing information view company websites.

<b>Hosiery Kits, Class One &amp; Two Hosiery</b>
Medi—www.mediuk.co.uk
Lohmann-rauscher (formally Activa) 222.lomann-raucher.co.uk
<b>Hosiery Wraps</b>
<b>Juzo</b> www.juzo.com

## Compression Bandages

Compression bandage kits	Sizes (ankle circumference)	NHS Logistics Code	Pack Size
<b>K-Two</b>	18-25cm 25-32cm	ECA152 ECA164	Single Single
<b>K-Two Reduced</b>	18-25cm 25-32cm	ECA205 ECA206	Single Single
<b>K-Four</b>	18-25cm 25-30cm Greater than 30cm	EVN008 EVN155 EVN156	Single Single Single
<b>Short Stretch Compression</b>			
<b>Actico</b>	6cmx6m 8cmx6m 10cmx6m 12cmx6m	EBA031 EBA032 EBA016 EBA033	Single Single Single Single
<b>K-Flex</b> Contains latex			

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>KCI V.A.C.</b> Granufoam Dressing Kit- Black Foam	Small	ELZ188 ELZ188/10	Single 10
	Med	ELZ197 ELZ197/10	Single 10
	Large	ELZ201 ELZ201/10	Single 10
White Foam Dressing Kit	Small	ELZ039	5
	Small	ELZ020	10
	Large	ELZ040	5
	Large	ELZ021	10
White foam only	Large	ELZ185	10
V.A.C. canisters	300ml (activac)	ELZ205	5 & 10
<b>Venturi Dressings</b>	Flat drain	ELZ136	Single
	Channel drain	ELZ137	Single
	Large Flat drain	ELZ138	Single
<b>Venturi Canisters</b>	600ml (Avanti)	ELZ139	Single
	300ml (Compact) (349-5991)	£12.24	Single

## Specialist Therapies

Negative pressure wound therapy (NPWT) to be applied only by staff who have undergone training. Please contact Tissue Viability for guidance.

## Negative Pressure Wound Therapy (NPWT) Therapy and Dressings

NPWT is for the management of heavily exuding large wounds or wounds that are difficult to heal. Funding of the NPWT equipment must be secured via the Tissue Viability Team prior to the commencement of treatment. (For further information refer to the NPWT Therapy Guidelines).

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>KCI V.A.C.</b> Granufoam Dressing Kit- Black Foam	Small	ELZ188 ELZ188/10	Single 10
	Med	ELZ197 ELZ197/10	Single 10
	Large	ELZ201 ELZ201/10	Single 10
White Foam Dressing Kit	Small	ELZ039	5
	Small	ELZ020	10
	Large	ELZ040	5
	Large	ELZ021	10
White foam only	Large	ELZ185	10
V.A.C. canisters	300ml (activac)	ELZ205	5 & 10

## Compression Bandages continued...

Compression Bandage Components	Sizes	NHS Logistics Code	Pack Size
<b>Ko-Flex Long</b> contains latex	10cmx7m	ECD028	18
<b>K-Plus</b> latex free	10cmx8.7m	ECA162	24
<b>K-Plus Long</b> latex free	10cmx10.25m	ECA172	24
<b>K-ThreeC</b> latex free	10cmx3m	EBA060	Single

## Tapes

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Clinipore</b> Permeable Non-Woven Synthetic Adhesive Tape	1.25cm x 10m	EHU019	24
	2.5cm x 10m	EHU020	12
	5cm x 10m	EHU021	6
<b>Hypafix</b> Permeable Apertured Non- Woven Synthetic Adhesive Tape	2.5cm x 10m	EHR030	Single
	5cm x 10m	ENH111	Single
	10cm x 10m	EHR113	Single
	15cm x 10m	EHR031	Single
	20cm x 10m	EHR117	Single
	30cm x 10m	EHR032	Single

## Zinc Paste Bandages

Indicated use: Management of dry and eczematous skin

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Viscopaste 5cm</b> band- age 10% zinc oxide	7.5cm x 6m	EFA011	Single
<b>ZipZoc</b> stocking 20% zinc oxide	17cm x 13cm	EFA018	Single

Proprietary Name	Size	NHS Logistics Code	Pack Size
<b>Appeel Sterile Adhesive remover</b>	5ml liquid sachet	ECX028	20
	Sterile Wipe	EXC029	30

### Non Adherent Primary Dressings

Indicated use: Lightly exuding wounds. Low adherence to wound bed. Non absorbent. Can be a primary or secondary dressing.

Proprietary Name	Sizes	NHS Logistics Price	Pack Size
<b>N/A Ultra</b>	9.5cm x 9.5cm	EKG031	40
	19cm x 9.5cm	EKG033	25
	19cm x 19cm	EKG035	5
<b>Atrauman</b>	5cm x 5cm	EKA024	50
	7.5cm x 10cm	EKA032	50
	10cm x 20cm	EKA036	30
	20cm x 30cm	EKA016	10

### Low Adherent Dressings

Indicated use: Lightly exuding wounds without necrosis. Protect wound bed from direct contact of secondary dressing. Can be a primary or secondary dressing.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Telfa</b>	5cm x 7.5cm	EJE051	100
	10cm x 7.5cm	EJE053	100
	15cm x 7.5cm	EJE055	100
	20cm x 7.5cm	EJE057	100

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Flaminal Hydro</b> light /moderately exuding wounds	15g	ELG021	5
<b>Flaminal forte</b> moderate/heavily exuding wounds	15g	ELG022	5
<b>Cutimed Sorbact</b> Absorbent dressing pad	10 cm x 10 cm 10 cm x 20 cm	ELY215 ELY219	5 20
<b>Cutimed Sorbact</b> swabs	4cm x 6cm 7cm x 9cm	ELY212 ELY213	5 5
<b>Cutimed Sorbact</b> ribbon	2cm x 50cm 5cm x 200cm	ELY218 ELY217	20 10

Proprietary Name	Size	NHS Logistics Code	Pack Size
<b>Debrisoft</b> for mechanical debridement, moisten before use	10cm x 10cm	ELZ354	5

## Specialist Dressings – Antimicrobials

Silver Dressings- alginate/hydrofibre/ packing	Sizes	NHS Logistics Code	Pack Size
<b>Aquacel Ag+ Extra Ribbon</b>	1cm x 45cm	ELY518	5
	2cm x 45cm	ELY519	5
<b>Aquacel Ag+ Extra</b>	5cm x 5cm	ELY514	10
	10cm x 10cm	EL7515	10
	15 x 15cm	ELY516	5

Proprietary Name	Size	NHS Logistics Code	Pack Size
<b>Surfactants Octenilin</b> NB: effective after minimum 30 seconds application	350 ml bottle	MRB443	Single
<b>Polymem</b> non-adhesive	10cmx10cm	ELA303	15
	15cmx15cm	ELA578	60
<b>Polymem Roll</b> Foam that contains cleansing agent	10cmx61cm	ELA321	4

## Absorbent Perforated Dressings With Adhesive Border

Indicated use: Lightly exuding wounds. Low adherence primary or secondary dressing.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Softpore</b>	6cm x 7cm	EIJ024	60
	10cm x 10cm	EIJ013	50
	10cm x 15cm	EIJ014	50
	10cm x 20cm	EIJ024	30
	10cm x 25cm	EIJ025	30
	10cm x 30cm	EIJ026	30
	10cm x 35cm	EIJ027	30

Indicated use: For light to moderately exuding wounds. Soft silicone wound contact dressing. Can be a primary or secondary dressing. (Alternative to Softpore for sensitive/fragile skin).

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Cutimed Siltec L (non border)</b>	5cm x 6cm	ELA439	10
	10cm x 10cm	ELA440	10
	15cm x 15cm	ELA441	10

## Vapour Permeable Adhesive Film Dressings

Indicated use: Superficial low exuding wounds and skin protection, IV dressing for securing and protecting cannulae. Partial thickness wounds with minimal exudate. Commonly used as secondary dressing. Use on fragile skin/those at risk of developing minor skin damage caused by friction or pressure.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Clearfilm</b>	6cm x 7cm	ELW646	100
	10cm x 12cm	ELW696	10
	12cm x 12cm	ELW707	10
	15cm x 20cm	ELW697	10
	20cm x 30cm	ELW698	10
<b>Tegaderm IV</b> Also for use with butterfly cannula	7cm x 8.5cm	ELW056	100
	8.5cm x 10.5cm	ELW074	50
<b>Hydrofilm plus</b> Adherent dressing film with pad	5cm x 7.2cm	ELW291	50
	9cm x 10cm	ELW292	25
	9cm x 15cm	ELW303	25
	10cm x 20cm	ELW304	25
	10cm x 25cm	ELW049	25
	10cm x 35cm	ELW250	25

## Antimicrobial Dressing

Indicated use: For infected wounds - Short-term use 7-14 days usually advised until infection resolves. For chronically colonised wounds - longer term use may be required. All require a secondary dressing.

Iodine dressings	Sizes	NHS Logistics Code	Pack Size
<b>Iodosorb</b>	10g ointment	EKB012	4
	3g powder	EKB010	7
<b>Iodoflex paste</b>	5g	EKB007	5
	10g	EKB008	3

Honey dressings	Sizes	NHS Logistics Code	Pack Size
<b>Medihoney</b> tube	20g	ELZ508	5
<b>Medihoney Tulle</b> Knitted viscose mesh impregnated with honey	10cm x 10cm	EJE067	5
<b>Medihoney Apinate</b> Alginate dressing impregnated with honey	5cm x 5cm	EJE076	10
	10cm x 10cm	EJE000	5



## Specialist Dressing Products

Only to be used under specialist advice, i.e. Tissue Viability Nurses, Tissue Viability Link Nurses and other Specialist Nurses and Doctors practising in the area of wound management.

### Specialist Primary Contact Dressings

Indicated use: Light to moderately exuding wounds. Coated with soft silicone which can be used on fragile skin. Beneficial to reduce frequency of primary dressing change.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Silflex</b>	5cm x7cm	EKH028	10
	8cm x 10cm	EKH029	10
	12cm x 15cm	EKH030	10
	20cm x 30cm	EKH032	10
<b>Urgotul</b>	5cm x 5cm	EKB056	10
	10cm x 10cm	EKB057	10
	15cm x 20cm	EKB058	10

### Silicone Wound Contact Dressing with Foam Film Backing

Indicated use: To absorb moderate amounts of exudate. Can be a primary or secondary dressing.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Allevyn Gentle Adhesive</b>	7.5cm x 7.5cm	ELA359	10
	12.5cm x 12.5cm	ELA361	10
	10cm x 20cm	ELA499	10
<b>Cutimed Siltec Non - adhesive</b>	5cm x 6cm	ELA446	10
	10cm x 10cm	ELA447	10
	15cm x 15cm	ELA449	10
	10cm x 20cm	ELA448	10

## Skin Protection/Barrier Products

Indicated use: Protection of peri-wound area and protection of skin from body fluids e.g. exudate and faeces, adhesives and tapes.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Medi-Derma S</b>	1 ml Foam Applicator	ELY532	25
<b>Medi-Derma S</b>	2g	ELY536	20
<b>Medi-Derma S</b>	100g	ELY538	Single
<b>Zinc &amp; Castor Oil Ointment</b> *** contains peanut oil	100g	MRB969	12
<b>Proshield Plus Skin</b> On specialist advice only	Protective 115g (9.94)	ELZ537	
<b>Proshield Foam &amp; Spray Skin Cleanser</b> On specialist advice only	235ml (£6.61)	ELZ536	

## Hydrogel Dressings

Indicated use: All tissue types, rehydration of necrosis and slough. For use on dry or lightly exuding wounds. Requires a secondary dressing.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
Hydrogel Aquaform	8g	ELG017	10
Hydrogel Sheet	5cm x 7.5cm	ELE072	5
Hydroorb	10cm x 10cm	ELE012	5

## Hydrocolloid Dressings

Indicated use: All tissue types, but only those with low to moderate exudate. Will rehydrate necrosis and slough.

**Promote granulation. Only use as a primary dressing.**

Proprietary Name	Sizes	NHS Logistics Code	Pack size
Duoderm Signal Dressing *	10cm x 10cm 14cm x 14cm	ELM079 ELM083	5 5
Tegaderm Hydrocolloid Dressing (Oval )	10cm x 12cm Pad 7cm x 9cm 13cm x 15cm Pad 10cm x 12cm	ELM084 ELM373	5 5
Duoderm Extra Thin *	4.4cm x 3.8cm 7.5cm x 7.5cm 5cm x 20cm 10cm x 10cm 15cm x 15cm	ELM051 ELM311 ELM319 ELM050 ELM051	10 5 10 10 10

\* This product contains pork gelatin

## Wound Manager Bags

**Indicated use: Heavily exuding open wounds. Can be left on up to 5 days.**

Proprietary Name	Sizes	NHS Logistics Code
Oakmed Wound Manager Pouch Refer to website for product guide.	Full range  www.oakmed.co.uk	See NHS Supply chain for code

## Pressure Ulcer Prevention

Indicated use: Gel pads that help redistribute pressure over bony areas. These can only be used on unbroken skin and can be cut to fit a specific area.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
Kerrapro Pads Heel	One size only	FES9911	2
Sacrum	One size only	FES9910	Single
Sheets	10cm x 10cm x 1.2cm 20cm x 20cm x 0.3cm	FES9913 FES9914	5 5

## Odour Absorbing Dressings

Indicated use: Malodorous wounds. Can be cut to fit the shape of the wound.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Clinisorb</b>	10cm x 10cm	ELV051	10
	10cm x 20cm	ELV053	10
	15cm x 25cm	ELV055	10

## Antimicrobial Dressings

Indicated use: For infected wounds - Short-term use 7-14 days usually advised until infection resolves. For chronically colonised wounds - longer term use may be required. All require a secondary dressing.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Inadine</b>	5cm x 5cm	EKB101	25
	9.5cm x 9.5cm	EKB502	25
<b>Atrauman Ag</b> Non- adherent mesh impregnated with metallic silver	5cm x 5cm	EKB039	10
	10cm x 10cm	EKB040	10
	10cm x 20cm	EKB041	10

## Alginate Dressings

Indicated use: To absorb moderate amounts of exudate. Requires a secondary dressing, promotes haemostasis.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>ActivHeal</b>	5cm x 5cm	ELS139	10
	10cm x 10cm	ELS141	10
	10cm x 20cm	ELS140	10

## Fibrous Dressings

Indicated use: To absorb moderate to heavy amounts of exudate when applied dry. Requires a secondary dressing. Do not wet prior to application.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Aquacel ribbon</b>	1cm x 45cm	ELY368	5
	2cm x 45cm	ELY013	5
<b>Aquacel Extra</b>	5cm x 5cm	ELY377	10
	10cm x 10cm	ELY378	10
	15cm x 15cm	ELY379	5
	4cm x 10cm	ELY489	10
	4cm x 20cm	ELY490	10
	4cm x 30cm	ELY491	10

## Simple Foam Dressings

Indicated use: To absorb low amounts of exudate. Can be a primary or secondary dressing.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Allevyn</b> non-adhesive	5cm x 5cm	ELA129	10
	10cm x 10cm	ELA131	10
	10cm x 20cm	ELA101	10
	20cm x 20cm	ELA133	10
<b>Allevyn</b> adhesive	7.5cm x 7.5cm	ELA020	10
	10cm x 10cm	ELA113	10
	12.5cm x 12.5cm	ELA024	10
	17.5cm x 17.5cm	ELA024	10
	12.5cm x 22.5cm	ELA046	10

## Tracheostomy Dressing

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
Permafoam Trachi-Dress	8cm x 8cm (fenestrated)	ELA266	10

## Basic Dressing Pad

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Bastos Viegas</b>	10cm x 10cm	EJA238	25
	10cm x 20cm	EJA080	25
	20cm x 20cm	EJA081	15
	20cm x 40cm	EJA082	10

### For FP10

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Xupad (sterile)</b>	10 x 12 cm	EJA092	25
	10 x 20 cm	EJA093	25
	20 x 20 cm	EJA094	15
	20 x 40 cm	EJA095	8

## Super Absorbent Dressings

Indicated use: To absorb moderate to heavy amounts of exudate. Can be a primary or secondary dressing.

Proprietary Name	Sizes	NHS Logistics Code	Pack Size
<b>Non Adherent Eclipse</b>	10cm x 10cm	EJE029	20
	15cm x 15cm	EJE032	20
	20cm x 30cm	EJE030	20
<b>Boot Shape</b>	47cm x 55cm	EJE085	5
	33cm x 48cm	EJE086	5
<b>Adherent Sorbion Sachet Border</b>	10cm x 10cm	EJE151	5
	15cm x 15cm	EME103	5
	15cm x 25cm	EME104	5